

**Camp Leo Basic Pump Training Resource Sheet**  
**Alphabetical listing of insulin pumps by manufacturer**  
**Jennifer Okemah MS RD BCADM CDE CSSD**

**Animas Ping**

- ▶ Current basal rate -> Home Screen
- ▶ Temporary basal rate -> Home screen if on temp basal. Tells basal change and time left.
  - ▶ To set temp basal: Home Screen ->highlight 'Menu' ->'OK' ->scroll down to 'Basal', hit 'OK' ->'temp' will be highlighted. Hit 'OK' ->'change %' will be blinking. Hit 'OK' -> make necessary increase or decrease. Hit 'OK' -> adjust duration ->hit OK ->hit 'Go'
- ▶ IOB – home screen
- ▶ Last bolus given – Home screen, highlight 'Menu' ->'OK' -> scroll down to 'History'. Hit 'OK' ->'Bolus' will be highlighted. Hit 'OK' ->this is last time and amount of bolus

**Animas Vibe**

- ▶ Current basal rate – main screen
- ▶ Temporary basal rate – Menu -> Basal -> Temp
  - ▶ Same as PING
- ▶ IOB
  - ▶ Home screen
- ▶ Last bolus given
  - ▶ Main screen -> History -> bolus

**Medtronic 522 or 530g**

- ▶ Current basal rate – Press 'ESC' key for current basal rate
- ▶ Temporary basal rate – open circle on main screen indicates a unique situation is occurring such as 'temp basal'. Press 'ESC' key for status of temp basal.
  - ▶ To set temp basal – Press 'ACT' ->scroll down to 'Basal'. Press 'ACT' ->press 'set/edit temp basal' – set duration of time –press 'ACT' – set % or u/hr change as instructed -> press 'ACT' - you should now see an open circle in upper left corner of home screen, right next to the "syringe" indicating reservoir volume.
- ▶ IOB – main menu, Bolus, Bolus History
- ▶ Last bolus given – Press 'ESC' key for last bolus given

**Medtronic 630g/670g**

- ▶ Current basal rate – highlight 'Basal' on home screen ->press center 'confirm' button. Current basal rate will show

- ▶ Temporary basal rate – highlight ‘Basal’ on home screen ->press center ‘confirm’ button. Scroll down to Temp Basal. Set duration of time ->‘confirm’. Change percent of basal per instruction. Hit ‘Begin’.
- ▶ IOB – called “active insulin” – on Home screen, bottom right or under “bolus” on home screen
- ▶ Last bolus given – Home screen. Bolus will be highlighted. ‘Confirm’. Last bolus will show. Also can access it by Home screen, highlight top bar using up key. Hit ‘confirm’. Scroll down to ‘quick status’. Last bolus, current basal is on there.

## Omnipod

- ▶ Current basal rate - Status screen - activate by pressing any button, then confirm identity of camper by name.
- ▶ Temporary basal rate - if active, will be on Status screen. To activate:
  - ▶ Press “home” button once PDM is in Status Screen. Scroll down to Temp Basal. Select “Increase” or “Decrease” per instruction. Select “next”. Enter duration of time by scrolling up. ‘Confirm’
- ▶ IOB - Status screen
- ▶ Last bolus given - Status screen

## Tandem

- ▶ Activate pump by touching sequences 1,2,3
- ▶ Current basal rate - OPTIONS, My pump, Personal Profiles, Edit to view but don't edit.
- ▶ Temporary basal rate - Options, Temp Rate, set % and duration, hit “start”.
- ▶ IOB - main screen
- ▶ Last bolus given - units and time remaining is on main screen

## CGM:

Note that sensor glucose (SG) may not be the same value as blood glucose (BG). There is a lag in sensor data. When in doubt; obtain a finger stick.

Sensor arrow direction should be noted and reported when therapeutic decisions are warranted